



PARASURE

PARAMETRIC ASSURANCE SYSTEMS

Class 6 Emulating Indicators

STOP!

be sure...be ParaSure

EMULATING INDICATOR



Class 6
3.5 min. at 134 °C

PCN: PS1110

EN-ISO 11140-1 (class 6 indicator)

Sterilizing conditions
turn indicator from
light blue to dark.

Manufactured in EU

PARASURE

LOT 121015

STEAM

Unprocessed indicator
Large circle is pale blue

EMULATING INDICATOR



Successfully processed indicator

Large circle is as dark or darker than small circle

PASS

Items in tray/pack
can be safely
used

EMULATING INDICATOR



Unsuccessfully processed indicator

Large circle is paler than small circle
Cause: only temperature present

FAIL

Do NOT use items
in tray/pack

EMULATING INDICATOR



Unsuccessfully processed indicator

Only partial colour change in large circle
Cause: insufficient steam penetration

FAIL

Do NOT use items
in tray/pack

EMULATING INDICATOR



Unsuccessfully processed indicator

Evidence of original pale blue colour
Cause: non-condensable gases present

FAIL

Do NOT use items
in tray/pack

PS1110 3.5 minutes at 134°C
PS1120 4.0 minutes at 134°C
PS1130 5.3 minutes at 134°C/15 minutes at 121°C

PS1140 7.0 minutes at 134°C/20 minutes at 121°C
PS1150 18 minutes at 134°C