




PARASURE

PARAMETRIC ASSURANCE SYSTEMS

Steam Sterilization Load Controls

STOP!

be sure...be ParaSure

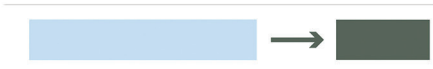

3.5 min. at 134 °C Sterilizing conditions turn indicator from light blue to dark.
PCN: PS1410

LOT 121018
STEAM ©PARASURE

Unprocessed indicator
Large square is pale blue



Successfully processed indicator

Large square is as dark or darker than small square

PASS
Items in tray/pack can be safely used



Unsuccessfully processed indicator

Large square is paler than small square
Cause: only temperature present

FAIL
Do NOT use items in tray/pack



Unsuccessfully processed indicator

Only partial colour change in large square
Cause: insufficient steam penetration

FAIL
Do NOT use items in tray/pack



Unsuccessfully processed indicator

Evidence of original pale blue colour in large square
Cause: non-condensable gases present

FAIL
Do NOT use items in tray/pack

PS1400 EN 867-5 Helix Device
PS1410 3.5 minute Load Control
PS1420 4.0 minute Load Control

PS1430 5.3 minute Load Control
PS1440 7.0 minute Load Control
PS1450 18.0 minute Load Control