

ParaSure CEI Cleaning Efficacy Indicator

- For monitoring the cleaning efficiency of washer-disinfectors
- Test soil simulation of human blood in accordance with ISO 15883
- Diagnostic - Easy to use, Easy to read
- Stainless steel - Non toxic
- Routine monitor/validation tool
- For use in Washer Disinfectors and Ultrasonic machines



ParaSure CEI (Cleaning Efficacy Indicator) cleaning efficiency check is designed for the validation and the routine monitoring of the cleaning efficacy of cleaning processes in washer disinfectors for surgical instruments.

- ParaSure CEI a special cleaning indicator made of stainless steel, which is contaminated with a quantitative measure of test soil (blood and additives), according to ISO 15883.

- This product is easy to use, easy to read and can be disposed of with domestic waste.

Order Details

PS1600
Cleaning Efficacy
Indicators
Box of 50

TECHNICAL SPECIFICATIONS



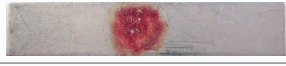
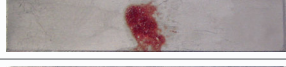

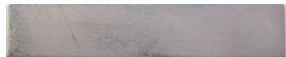
Name:	ParaSure CEI (Cleaning Efficacy Indicator)
Classification:	ISO 15883
Low Storage conditions/RH:	+18°C - RH 35%
High Storage conditions/RH:	+25°C - RH 70%
Lot no.	yes
Prod date:	yes
Exp date:	yes
Correct symbols:	in accordance with ISO 15883
Indicator:	Cleaning Efficacy Indicator
Disp/single use:	yes
Extra information:	
Non Toxic	yes
Shelf Life	18 months

ParaSure CEI Cleaning Efficacy Indicator



PARASURE

Assessment-Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
INSUFFICIENT CLEANING PROCESS:		
	Spot of test soil completely visible	Do not release batch! Check load, temperature, dosage and time. Repeat test. In case of repeated insufficient cleaning process: Call service technician.
	Spot of test soil visible	
	Spot of test soil visible	
	Spot of test soil slightly visible	
SUCCESSFUL CLEANING PROCESS:		
	Spot of test soil completely removed	Release batch. No further action needed.

Address: Squirrels | Kithurst Park |
Storrington | West Sussex | RH20 4JH | England |
Tel: +44 (0) 1903 741390

Warehouse Address: Units B10/B11
Oakendene Estate | Bolney Road | Cowfold |
West Sussex | RH13 8AZ | England