

Standard Basket Process Challenge Device (PCD)

- Stainless steel tray for testing of the ParaSure CEI
- Conforms to ISO 15883
- Easy to use – Easy to read
- Non toxic
- For use in Washer Disinfectors and Ultrasonic machines



Standard PCD (Washer/Disinfector Process Challenge Device)

- Container system for the ParaSure CEI (Cleaning Efficacy Indicator) creating a realistic and repeatable challenge. Process Challenge Device (PCD) for standard/general non-lumen instrumentation.

Order Details

PS1610

Standard Basket Process Challenge Device (PCD)


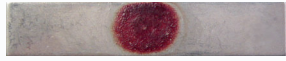
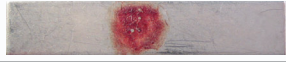
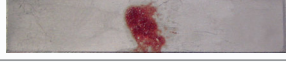

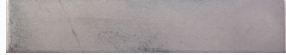
Each

Standard Basket Process Challenge Device (PCD)

TECHNICAL SPECIFICATIONS	
Name:	Standard Basket Process Challenge Device (PCD)
Classification:	ISO 15883
Storage conditions:	n/a
RH:	n/a
Lot no.	no
Prod date:	n/a
Exp date:	n/a
Correct symbols:	n/a
Indicator:	In conjunction with the ParaSure CEI
Disp/single use:	resuable
Extra information:	yes
Non Toxic	



Assessment-Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
INSUFFICIENT CLEANING PROCESS:		
	Spot of test soil completely visible	Do not release batch! Check load, temperature, dosage and time. Repeat test. In case of repeated insufficient cleaning process: Call service technician.
	Spot of test soil visible	
	Spot of test soil visible	
	Spot of test soil slightly visible	
SUCCESSFUL CLEANING PROCESS:		
	Spot of test soil completely removed	Release batch. No further action needed.

Address: Squirrels | Kithurst Park |
 Storrington | West Sussex | RH20 4JH | England |
 Tel: +44 (0) 1903 741390

Warehouse Address: Units B10/B11
 Oakendene Estate | Bolney Road | Cowfold |
 West Sussex | RH13 8AZ | England