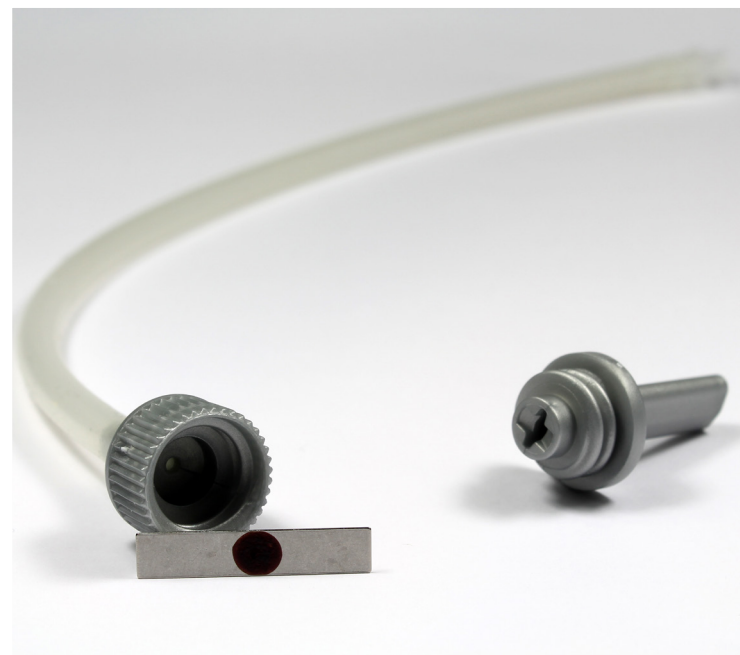


Standard Lumen PCD

- Unique simulation of cannulated instruments
- Conforms to recommendations in EN ISO/TS 15883-5
- Realistically challenges the process
- Easy to interpret results
- Polypropylene head
- Cost effective



The ParaSure Standard Lumen Process Challenge Device (PCD) simulates the cannulated devices used in Minimally Invasive Surgery (MIS), to effectively test the machine's cleaning performance. The polypropylene head houses the indicator and is connected to a 30 centimetre length of PTFE tubing which terminates in a luer lock.

Manufactured in accordance with EN ISO/TS 15883-5:2005, the Standard Lumen PCD challenges the machine to ensure effective soil removal by simulating the conditions found within cannulated instruments.

The Standard Lumen PCD is designed to be used with the ParaSure Cleaning Efficacy Indicators (CEI), PS1600.

Order Details

PS1620P

Standard Lumen PCD
each



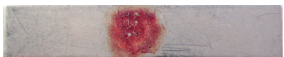
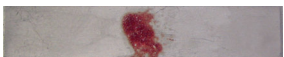


TECHNICAL SPECIFICATIONS

Product name:	Standard Lumen PCD
Unit of measure:	Each
Conforms to:	EN ISO/TS 15883-5:2005
Materials:	Polypropylene and PTFE
Frequency of use:	Every cycle
Number of uses:	Reusable
Indicator:	ParaSure Cleaning Efficacy Indicators (PS1600)
Supplied with:	Instructions for Use & Assessment Aid



PARASURE

Assessment-Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
INSUFFICIENT CLEANING PROCESS:		
	Spot of test soil completely visible	Do not release batch! Check load, temperature, dosage and time. Repeat test. In case of repeated insufficient cleaning process: Call service technician.
	Spot of test soil visible	
	Spot of test soil visible	
	Spot of test soil slightly visible	
SUCCESSFUL CLEANING PROCESS:		
	Spot of test soil completely removed	Release batch. No further action needed.

For further information about any of our products, please visit:

www.parasure.co.uk

ParaSure Limited

Office: Squirrels | Kithurst Park | Storrington | West Sussex | RH20 4JH | England
 Warehouse: Units B10/B11 | Oakendene Estate | Bolney Road | Cowfold | West Sussex | RH13 8AZ
 Telephone: +44 (0) 1903 741390 | Email: info@parasure.co.uk