



# PARASURE

## ParaSure DentaClean

For the rapid detection of protein residues in endoscope

- Cost effective way of monitoring soil removal
- Conforms to recommendations in HTM 01-05
- Cleaning Efficacy Indicator conforms to EN ISO 15883
- Realistically challenges the process
- Easy to interpret
- Instant result
- Supplied as 1 polypropylene basket
- and 12 Cleaning Efficacy Indicators



### ParaSure DentaClean

- ParaSure DentaClean offers a unique and cost effective option for all dental practices to ensure their decontamination process and soil removal efficacy is in accordance with HTM 01-05. The DentaClean polypropylene basket simulates a bur basket inside which is placed a ParaSure Cleaning Efficacy Indicator.

Used once a month, the DentaClean system provides a cost effective way of ensuring the washing machine is functioning correctly by removing all soil from the indicator. The diagnostic technology gives a clear and instant result and is supported by a comprehensive reference guide. ParaSure DentaClean is supplied as one device with 12 indicators offering a year's supply for each washing machine.

Order Details

**PS1750**

DentaClean



[WWW.PARASURE.CO.UK](http://WWW.PARASURE.CO.UK)



[WWW.PARASUREORGANICS.COM](http://WWW.PARASUREORGANICS.COM)

#### Registered Address:

ParaSure Limited Units B10/11 Oakendene  
Estate Bolney Rd Cowfold West Sussex RH13  
8AZ England

+44 (0)1403 864777

Ref: ParaSure Tech Sheets 02/13 Rev.D



# PARASURE

## ParaSure DentaClean

### TECHNICAL SPECIFICATIONS

<b>Name:</b>	DentaClean
<b>Classification:</b>	ISO 15883
<b>Storage conditions:</b>	n/a
<b>RH:</b>	n/a
<b>Lot no.</b>	n/a
<b>Prod date:</b>	n/a
<b>Exp date:</b>	n/a
<b>Correct symbols:</b>	n/a
<b>Indicator:</b>	In conjunction with the ParaSure CEI
<b>Disp/single use:</b>	reusable
<b>Extra information:</b> Non Toxic	yes

### Assessment-Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
<b>INSUFFICIENT CLEANING PROCESS:</b>		
	Spot of test soil completely visible	Do not release batch!  Check load, temperature, dosage and time.  Repeat test.  In case of repeated insufficient cleaning process: Call service technician.
	Spot of test soil visible	
	Spot of test soil visible	
	Spot of test soil slightly visible	
<b>SUCCESSFUL CLEANING PROCESS:</b>		
	Spot of test soil completely removed	Release batch. No further action needed.

For further information about any of our products, please visit:  
[www.parasure.co.uk](http://www.parasure.co.uk)

#### Registered Address:

ParaSure Limited Units B10/11 Oakendene  
 Estate Bolney Rd Cowfold West Sussex RH13  
 8AZ England

+44 (0)1403 864777



[WWW.PARASURE.CO.UK](http://WWW.PARASURE.CO.UK)



[WWW.PARASUREORGANICS.COM](http://WWW.PARASUREORGANICS.COM)